

Start Here First



Need Help?

For assistance please call the TheraSense Customer Service toll free number: 1-888-522-5226 (24 hours a day).

For detailed information about setting up your FreeStyle Tracker™ Diabetes Management System, refer to the *System Installation Guide*.



© 2002 TheraSense, Inc. All rights reserved.

FreeStyle Tracker is among the trademarks owned by TheraSense, Inc. HotSync and the HotSync logo are registered trademarks of Palm, Inc. Handspring, Visor, and the Handspring Logo are trademarks of Handspring, Inc.

ART01844 Rev. B

FreeStyle
TRACKER™
Diabetes Management System

System Contents Step-by-Step Instructions

- 1 First open the Handspring™ Visor™ package.*



- 2 Locate the PDA, remove its screen cover and install the batteries or charge the PDA.*



- 3 Set up the basic configuration of the new PDA (date, time, etc.)*




- 4 Plug the HotSync® Cradle connector into the PC USB or serial port.*



- 5 Install the Palm Desktop software by inserting the Handspring™ CD (supplied in the Visor package) into the PC's CD drive.*



- 6 Ensure that the HotSync® Manager is running on the PC, identified by this icon on the taskbar.*

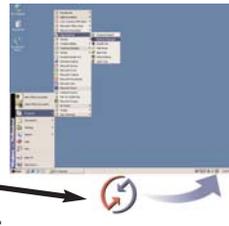

- 7 Put the PDA in its Cradle and press the HotSync® button. When the synchronization is finished, remove the PDA from its Cradle.




- 8 Open the FreeStyle Tracker™ Module and Software package.



- 9 Locate the FreeStyle Tracker™ Software CD and insert the CD into the PC's CD drive to install the software on the PC.




- 10 In the "Pilot Install" screen select USB or COM port as appropriate. **Install the FreeStyle Tracker software** on your PDA by pressing the HotSync button.

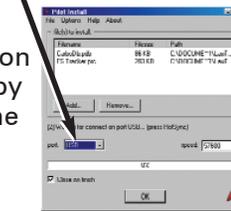

- 11 When the synchronization is finished, remove the PDA from its Cradle.




- 12 Follow the prompts on your PC to complete the software installation.

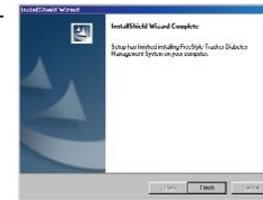

- 13 Tap the  (FS Tracker) icon on the PDA, tap the  (Menu) icon, and then tap Preferences.



- 14 Tap each item in the Preferences menu, then make the desired entries. (To make entries in the Prescribed Plan table, tap each numerical value in the table to display its entry screen).

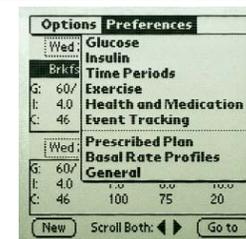

* You may have already performed these steps if you have been using your PDA prior to the FreeStyle Tracker System setup.

For more detailed instructions for each of the steps above, refer to the *System Installation Guide*.